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Versailles Grand Parc

1 avenue de l'Europe

78117 Toussus-le-Noble - France

N° Service Client +33(0)1 8377 0000

www.natureetdecouvertes.com

NATURE
DECOUVERTE

CEINTURE LOMBAIRE CHAUFFANTE

BEHEIZBARER NIERENGURT

LUMBAR HEAT BELT

Réf. 15205530

NOTICE

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TECHNICAL INFORMATION

Model	HP-007A
Poids	0,44 kg
Rated power	100W
Rated voltage	220-240V / 50Hz
	Indoor use only
	The device is equipped with double protective insulation and corresponds to protection class II
	After its service life, please dispose of this device according to the directive on Waste Electrical and Electronic Equipment 2002/96/EC (WEEE)

Technical and design changes in the course of continuous product development reserved.

INFORMATION ON COMPLIANCE WITH GUIDELINES

The manufacturer hereby declares that the device HP-007A complies with the fundamental requirements of the European Directive on Electromagnetic Compatibility (2014/30/UE) and the Low Voltage Directive (2014/35/UE).

- Do not use any strong cleaning agents or solvents.
- Please note that the cover may not be chemically cleaned, wrung out, dried in a tumble dryer or ironed.
- Please note that high stress is placed on the heating pad cover by frequent washing. The cover should be washed in a washing machine a maximum of five times during its service life. We therefore recommend that you wash the cover exclusively by hand.

3. Immediately after washing, pull the cover into its original shape and leave it to dry lying flat over a washing stand.

WARNING!

- Do not use any clothes pegs or similar to attach the heating pad cover to the washing stand. Otherwise this may damage the cover.

4. Let the cover dry completely before putting it back over the heating pad.

WARNING!

- Only reconnect the plug to the heating pad when the plug connection and the cover are both completely dry. Otherwise this may damage the heating pad.
- Never switch the heating pad on to dry it. This may cause an electric shock.

Step switch

Only clean the step switch with a dry cloth.

STORAGE AND PERIODIC INSPECTION

1. Pull the plug out of the socket.
2. Let the device cool down completely before storing it away.

WARNING!

- Store the product in a dry place.
- Avoid bending the device being bent sharply by items being placed on it during storage. technicians. Never open the device. The individual components of this device may not be disassembled, repaired or modified.
- Regularly inspect the device and power cord for possible signs of wear or damage.
- Please return the device to your point of sale in case of any wear or malfunction.
- Repairs must be performed exclusively by authorized customer service technicians. Never open the device. The individual components of this device may not be disassembled, repaired or modified.

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4. Step switch

You can choose the following settings:

Settings	Function
0	Off
1	Light heat
2	Medium
3	High

NOTE:

You can identify the setting using the indicator light.

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CLEANING AND CARE

- Electric underblankets can be washed on a delicate cycle at a maximum of 40°C.
- Bleaching not allowed.
- Do not dry electric underblankets in tumble dryers.
- Do not iron electric underblankets.
- No chemical cleaning.

COMPOSITION : Outer : 100 % polyester, Lining: 100 % polyester.

Please note the care instructions on the labels.

Cover and heating pad

1. First remove the plug from the electricity supply.
2. You can wash the cover in the washing machine on a delicate cycle at 40°C.
Use a mild detergent and follow the instructions of the manufacturer on amount of detergent.

WARNING!

- Pull the plug out of the socket before each time you clean the device.
- Only clean the device when it is cool. Risk of burns!
- Do not submerge the heating pad with the step switch in water.
This could lead to an electric shock or a fire.

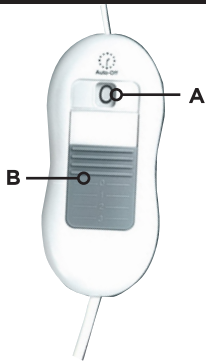
- If you dispose of the device at any one point in time. Please cut off the cable to make it useless.

In order to be able to utilize all the benefits of your electric warming device, please thoroughly read these instructions and observe the notes on care and safety. You will then be able to enjoy this device for a long time.

Please store these instructions for further use or in case you hand this device over to another person.

OPERATING ELEMENTS

A	Display screen
B	On / Off button

1. Electrical connection

To operate the lumbar heat belt, insert the plug into the mains power supply (insert the plug into a properly installed socket) and switch on the socket. Ensure that the device voltage (see label) and the mains voltage are the same.

2. Operation

The product should preferably be used on the following regions of the body: back.

WARNING!

Always use the heating pad in the cover. This creates a minimum distance which protects against overheating. Do not cover the step switch. This can lead to malfunctions.

3. Switching on

Move the step switch on the controller to the desired setting. The indicator light will illuminate to show that the device is in operation.

NOTE:

Unusual smells may occur during the first use. These are harmless and will cease over the course of operation.

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- **Examine the appliance frequently for signs of wear or damage. If there are such signs or if the appliance has been misused, return it to the supplier prior to any further uses.**
- **If the supply cord is damaged, it must be replaced by the manufacturer, its service agent or smiliarly qualified persons in order to avoid a hazard.**

- **Do not insert any needles or pointy objects into the device.**

UTILISATION

Thank you for choosing our product. We hope you enjoy the device.

- The device is intended exclusively for domestic use, not for commerical applications or the use in hospitals or medical offices.
- Please consult a physician prior to using the device if you suffer from an illness (e.g. varicose veins, thrombosis, etc.)
- Never use the device with open wounds, burns, hematoma, swellings, or similar conditions.
- Stop treatment if it feels uncomfortable to you. Heat-sensitive persons in particular should use the device with care and stop the treatment in a timely manner.
- Never use the heat device on animals.
- Use the device only in dry rooms and with a dry body-never in the bathtub, shower, above filled sinks in pools, saunas, or outdoors.
- Never operate the device too close to other heat sources.
- Do not pull the power cord across sharp edges.
- Do not pull on the cord or the device to disconnect the power plug or transport the device.
- Disconnect the power plug if the device is not in use.

safely.

- **This device uses supply voltage. Therefore, the same safety regulations apply as for any other electric device.**
- **Operate the device only with the stated supply voltage.**
- **Used only with detachable controller (GL-01).**
- **In order to ensure the proper functionality of the operating element (thermostat), do not place it on top of or underneath the device.**

• **Never use the heat device if it is folded wrinkled, or pushed together.**

- **Inspect that all components of the device are in a faultless condition prior to each start-up of the device. In the event of a defect or a malfunction, switch the device off immediately and disconnect the power plug from the power network.**
- **When the product is not being used, stow it away as instructed.**
- **Do not crease the product by placing objects on it during storage.**
- **Do not wrinkled the device. Fold the product carefully for storage.**
- **Store the device at a dry location if it is not in use.**
- **Let the device cool off prior to folding or rolling it.**

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PLEASE CAREFULLY READ THIS MANUAL AND KEEP IT IN A SAFE PLACE.

SAFETY INFORMATION

- This product must not be used by persons insensitive to heat or by other vulnerable persons incapable of reacting in the event of overheating

- Children under three years of age must not use this product due to their inability to react in the event of overheating.

- This appliance can be used by children aged from 8 years and above and person with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved. Children shall not play with appliance. Cleaning and user maintenance shall not be made by children without supervision.
- Please ensure that children do not use the device as a toy.
- The product must not be used by young children over three years of age unless the controls have been preset by a parent or guardian or if the instructions have been explained to the child in such a way that they can operate the controls

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