## BLOSSUM MASK BAG USER MANUAL

1. PRODUCT DESCRIPTION
2. CONTENT
3. INSTRUCTIONS BEFORE USE
4. INSTRUCTIONS FOR USE
5. WARRANTY
6. ENVIRONMENT PROTECTION
7. DECLARATION OF CONFORMITY
8. TO PROTECT YOURSELF AND OTHERS FROM CORONAVIRUS

### 1. PRODUCT DESCRIPTION:


The BLOSSUM MASK BAG is a textile heating bag, whose function is to decontaminate used masks. The heating cycle deactivates certain viruses\* and bacteria\*\*, which do not survive at high temperatures for a certain period of time. Once decontaminated, you will be able to reuse your mask.

\*Test performed by a French independent laboratory on disposable masks contaminated by viral strain human coronavirus HCoV-229 E. Results: reduction in the viral load of the human Coronavirus strain 229E by more than 6 log (i.e. more than 99.9999% reduction).

\*\*Test performed by Guangzhou Institute of Microbiology, independent lab accredited by CNAS and CMA, Escherichia Coli 8099 bacteria. Results: Killing rate = 99.94%

### 2. CONTENT :

The BLOSSUM MASK BAG includes a heating pouch with a USB power cord.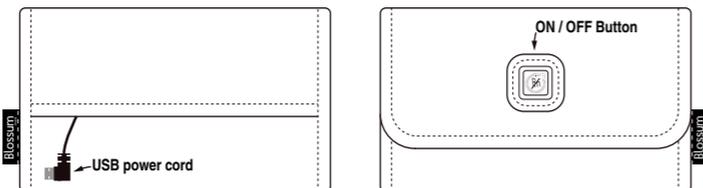

### 3. INSTRUCTIONS BEFORE USE

It is essential to read the instructions carefully before using the product and to keep them for future reference. Please have it read by any other user.

The Blossum Mask Bag enables a mask to be reused without overriding the recommendations and rules for using it.

#### SAFETY INSTRUCTIONS

**When using electrical equipment, certain basic safety precautions must be followed to reduce the risk of fire, electric shock and/or injury to persons.**

**Failure to observe the following instructions may result in property damage or physical injury. Read these instructions in their entirety.**

1. This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or persons without experience or knowledge, unless they have been given supervision or prior instruction in the use of the appliance by a person responsible for their safety.
2. Children should be supervised to ensure that they do not play with the appliance.
3. Power cord: The cord length of this appliance has been set to avoid the risk of accidents. Never pull on the power cord of the unit. To unplug the power cord, remove the power plug from the power source without forcing it. To turn on the unit, connect the unit to a power source specified in this manual. Do not use an electrical appliance with a damaged cord or plug, or an appliance that is defective or damaged in any way. You must have it examined, repaired or have a mechanical or electrical adjustment made by a qualified technician to avoid any danger. Check the condition of the power cord regularly. If the power cord is damaged, it must be replaced by the manufacturer, its after-sales service or a person of similar qualification to avoid any danger.
4. To reduce the risk of electric shock, do not immerse the cord, plug, or appliance in water or other liquids.

5. The use of accessories or equipment not recommended by the manufacturer, the reseller or the importer, may result in fire, electric shock, or injury to persons.
6. Do not use in a wet environment.
7. If the unit shows any signs of malfunction during use, unplug it immediately. Do not use or attempt to repair a defective unit.
8. Do not use the appliance if it has been dropped or submerged in water.
9. Do not bend the appliance, except when closing it (see operating instructions).
10. Do not operate the appliance with wet or damp hands.
11. Do not use the appliance for purposes other than those for which it has been designed.
12. Do not use the appliance with a power source other than specified in this manual (see power sources section).
13. Do not store the unit in a wet place. Air humidity and temperature variations cause water or condensation that can oxidize the electrical components in the unit.


### 4. INSTRUCTIONS FOR USE

#### User manual:

1. Insert the mask flat into the BLOSSUM MASK BAG.
2. Close the BLOSSUM MASK BAG using the 2 press studs (A). Do not fold the mask. Do not fold the BLOSSUM MASK BAG except to close the flap.
3. Connect the USB power cord (B) to one of the power sources provided in this manual.
4. Press the power button (C) without forcing it until the indicator light flashes (about 3 seconds). Once it flashes, the heating cycle starts.
5. The light flashes every four seconds for 30 minutes (to reach the required temperature). As soon as the light is steady, the BLOSSUM MASK BAG has reached the required temperature and the heating cycle continues.  
**If the temperature is too cold, the device may not reach the temperature required after 30 minutes. The indicator light will continue flashing at a rapid rate (0.5s off/0.5s on). You need to switch off the appliance. Repeat the operation when you are in a warmer place.**
6. As soon as the indicator light switches off, the cycle is complete. Disconnect the power source (B). Always pull the power cord at the plug, do not pull the cable itself.
7. Let the BLOSSUM MASK BAG cool down.
8. Make sure your hands are clean before touching the decontaminated mask.
9. Make sure the mask is cold before using it.

The decontamination cycle lasts 95 minutes:

- 30 minutes: to reach the required temperature level
- 60 minutes: high temperature heating cycle
- 5 minutes: cooling cycle

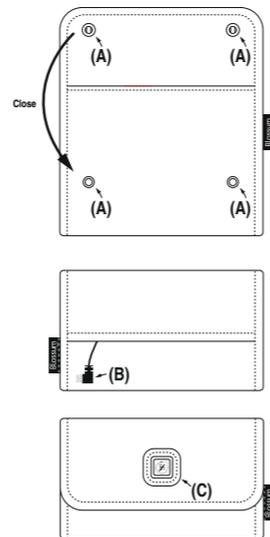### Precautions for use :

1. Put one mask at a time in the BLOSSUM MASK BAG.
2. Suitable for reusable/washable fabric face masks and disposable masks.
3. It is recommended not to decontaminate a disposable mask more than 4 times.
4. Check the condition of your mask after decontamination. Do not use it if it looks faulty.
5. If you use the Blossum Mask Bag to disinfect a mask that is moist, please ensure that it is fully dry before wearing once more. Do not use the Blossum Mask Bag with a wet mask.
6. If the indicator light does not switch on, or if the indicator light comes on and off quickly, check that the power source is compatible with the recommendations in this manual (see Power sources section).
7. If the indicator does not light up, or if the indicator lights up and goes out quickly, and if you are using an external battery check the charge of your external battery and use a charged battery.
8. If the light comes on and the bag does not warm up, or if the light does not come on and the external battery is charged, contact us via [www.blossum-mask.com](http://www.blossum-mask.com).
9. **DO NOT PUT YOUR HAND INSIDE THE BLOSSUM MASK BAG WHEN IT IS ON. RISK OF BURNS.**
10. Do not put heat sensitive products close to the appliance when heating.
11. We recommend you not to use the BLOSSUM MASK BAG in an environment where heat dissipating fast such as windy place, air-conditioning rooms, metal surface...

### POWER SOURCES:

**Do not connect to AC power.**

**Power supplies below 2A are not suitable.**

Power source features: DC 5V-9.5W 2A

Portable charger or Powerbank, Car cigarette-lighter, Laptop USB port - only the most recent models, USB wall-charger.

#### Cleaning instructions

1. Never immerse the unit in water or any other liquid.
2. Clean any parts that are dirty with a damp cloth without rubbing.
3. Let your unit dry completely before using it again.

### 5. WARRANTY

This appliance is warranted for 2 years, parts and labor, from the date of purchase against non-conformity, defect or manufacturing defect.

The warranty covers the replacement of any defective part but does not cover normal wear and tear, lack of maintenance, shocks or damage that may occur as a result of overvoltage or misuse.

It is strictly forbidden to repair the appliance yourself. Otherwise, the seller, the manufacturer and the importer reserve the right to refuse any guarantee.

Any intervention by the user on the appliance, or any modification made by the user to the device or its original accessories will result in the loss of the warranty.

Improper use, other than as described in these operating instructions, will result in forfeiture of the warranty.

### 6. ENVIRONMENT PROTECTION

After wear and tear, the packaging as well as the appliance must be dropped off at a municipal waste collection service to ensure the disposal of the product as well as material recovery (including energy recovery for plastics). If in doubt, please contact your local council, which will provide you with information on the collectors at your disposal. Compliance with these procedures encourages the recycling of the various components of WEEE (Waste Electrical and Electronic Equipment).

### 7. DECLARATION OF CONFORMITY

This appliance complies with the ROHS directive 2011/65/UE on restrictions on the use of hazardous substances contained in electrical and electronic equipment.

### 8. TO PROTECT YOURSELF AND OTHERS AGAINST CORONAVIRUS

- Wash your hands very often.
- When coughing or sneezing, cover your mouth and nose with your arm or use a disposable tissue.
- Dispose of any tissues you have used.
- Avoid close contact when greeting people.

#### How to wear a mask

- Wash your hands with soap before touching the mask.
- Hold the mask by the ear loops and place a loop around each ear.
- Tie the laces behind the head and neck.
- Mold or pinch the stiff edge to the shape of your nose.
- Pull the bottom of the mask over your mouth and chin.

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