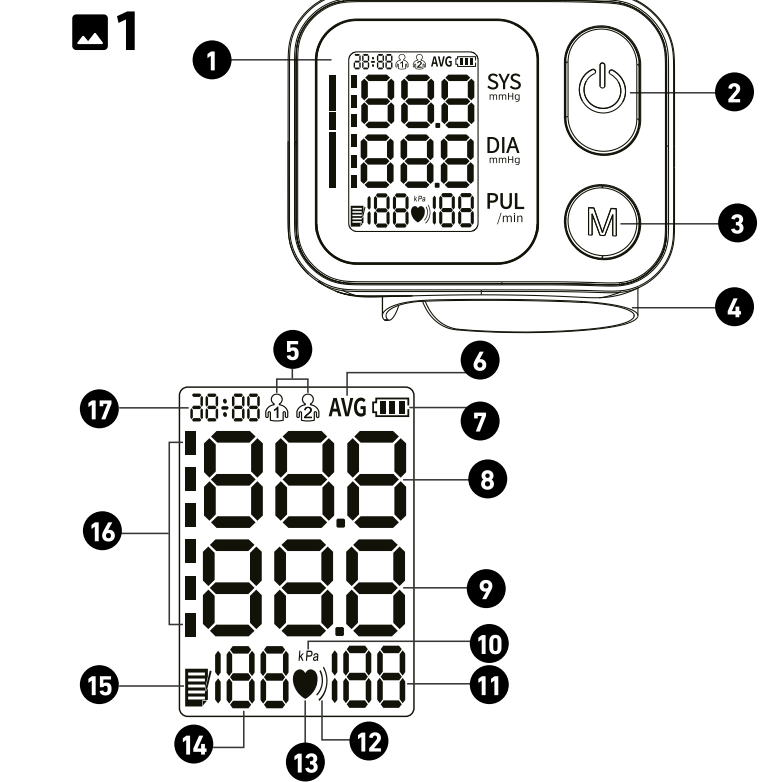
ENGLISH

PRESENTATION

Thank you for choosing the WBPM-200 from LANA-FORM.

Please read all the instructions before using your product, particularly these basic safety instructions.

The photographs and other images of the product in this manual and on the packaging are as accurate as possible, but do not claim to be a perfect reproduction of the product.

IMPORTANT INFORMATION

WHAT IS BLOOD PRESSURE?

Blood pressure is the force exerted by blood against the walls of the arteries. Systolic pressure occurs when the heart contracts and diastolic pressure occurs when the heart expands. Blood pressure is measured in millimeters of mercury (mmHg). Our natural blood pressure is represented by the fundamental pressure, which is measured first thing in the morning while we are still at rest and before eating.

WHAT IS HYPERTENSION AND HOW IS IT CONTROLLED?

Hypertension, an abnormally high arterial blood pressure, if left unattended, can cause many health problems, including stroke and heart attack. Hypertension can be controlled by altering lifestyle, avoiding stress and with medication under a doctor's supervision. To prevent hypertension or keep it under control: do not smoke, reduce salt and fat intake, exercise regularly, have regular physical checkups and maintain an appropriate weight.

WHY MEASURE BLOOD PRESSURE AT HOME?

Blood pressure measured at a clinic or doctor's surgery may cause apprehension and produce an elevated reading, 25 to 33 mmHg higher than that measured at home. Home measurement then reduces the effects of outside influences on blood pressure readings, supplements the doctor's readings and provides a more accurate, complete blood pressure history.

NORMAL BLOOD PRESSURE FLUCTUATION

All physical activity, excitement, stress, eating, drinking, smoking, body posture and many other activities or factors (including taking one's blood pressure) will influence blood pressure value. Because of this, obtaining identical multiple blood pressure readings is generally unusual. Blood pressure fluctuates continually, day and night. The highest value usually appears in the daytime and the lowest usually at midnight. Typically, the value begins to increase at around 3 am, and reaches its highest level in the daytime while most people are awake and active. In view of this information, it is recommended that you measure your blood pressure at approximately the same time each day.

4 PRECAUTIONS TO TAKE BEFORE USE

- If you are on medication, please consult your doctor for the appropriate time to take blood pressure readings. Never change prescriptions without consulting your doctor first.
- For those with irregular or unstable peripheral circulations such as liver disease, diabetes, hardening of the arteries, etc., measurement values might vary by position; there is a difference in values measured at the wrist from those of the upper arm.
- Measurements may be incorrect if the device is used near active mobile phones, television sets, microwave ovens, X-ray machines, and other devices with strong electrical fields. To prevent interference, please use this device away from such devices or turn them off before taking measurements.
- Always wash your hands before taking measurements.
- Do not take measurements on the same arm connected to equipment monitoring ME in order to avoid loss of function.
- Refer to the "Troubleshooting" section of this manual if abnormal readings are obtained and consult your doctor if necessary.
- Portable RF communications equipment (including peripherals such as antenna cables and external antenna) should be kept at least 30 cm from the device and its parts to avoid impaired performance.
- Measurement values at the hospital may be lower than those taken at home due to the difference in environment.

5 PRODUCT DESCRIPTION

Based on oscillometric methodology and a silicon integrated pressure sensor, blood pressure and pulse rate can be measured automatically and non-invasively. The LCD display will show blood pressure and pulse rate. The most recent 2x199 measurements can be stored in the memory with a date and time stamp. This device complies with the electromagnetic compatibility requirement of EN60601-1-2, safety standards of EN60601-1 and performance of IEC 80601-2-30 as specified in EEC directive 93/42/EEC.

1 FEATURES, CONTENT AND DISPLAY INDICATORS

- HD LCD screen
- ON/OFF button
- General settings button
- Wrist cuff
- User icon
- Average value icon
- Battery icon
- Systolic blood pressure value
- Diastolic blood pressure value
- Unit of pressure
- Pulse rate value
- Irregular heartbeat icon
- Heartbeat icon
- Memory number
- Memory icon
- Blood pressure classification
- Date & Time

Content of the box : WBPM-200, 2 x AAA batteries, user manual

Features of the WBPM-200 :

- 2 users, 199 memory sets for each user
- Large HD LCD screen
- Easy to use, one button needed to operate
- Auto power off within 20s without any operation
- WHO classification
- Detection of an irregular heartbeat
- Low battery indication
- Average reading of last 3 measurements

2 INTENDED USE

The WBPM-200 is for use by medical professionals or at home and is a non-invasive blood pressure measurement system intended to measure the diastolic and systolic blood pressure and pulse rate of an adult individual by using a non-invasive technique in which an inflatable cuff is wrapped around the wrist.

3 CONTRA-INDICATION

This appliance must not be used by a child or by a mentally-impaired individual on their own. This blood pressure monitor is designed for adults and should never be used on infants or young children.

It is inappropriate for people with serious arrhythmia to use this blood pressure monitor.

Prolonged over-inflation (cuff pressure exceeding 300 mmHg or maintained above 15 mmHg for longer than 3 minutes) of the bladder may cause ecchymosis of your arm.

15mmHg for longer than 3 minutes) of the bladder may cause ecchymosis of your arm.

Consult your physician if you have any doubts about the cases below :

- The application of the cuff over a wound or inflammation sites;
- The application of the cuff on any limb where intravascular access or therapy, or an arterio-venous (A-V) shunt, is present;
- The application of the cuff on the arm on the side of a mastectomy;
- Simultaneously used with other medical equipment used for monitoring purposes on the same limb;

This blood pressure monitor is designed for adults and should never be used on infants or young children. Consult your physician or other health care professionals before use on older children.

Do not use this unit in a moving vehicle; this may result in incorrect measurements.

Do not measure after physical exercise or taking a bath.

Blood pressure measurements determined by this monitor are equivalent to those obtained by a trained observer using the cuff/stethoscope auscultation method, within the limits prescribed by the American National Standards Institute for electronic or automated sphygmomanometers.

Information regarding potential electromagnetic or other interference between the blood pressure monitor and other devices together with advice regarding avoidance of such interference please see part ELECTROMAGNETIC COMPATIBILITY INFORMATION.

If Irregular Heartbeat (IHB) brought by common arrhythmias is detected during the procedure for measuring blood pressure, a signal (♥) will be displayed. Under this condition, the blood pressure monitor can keep functioning, but the results may not be accurate. You should then consult your physician for an accurate assessment.

There are 2 conditions under which the IHB signal will be displayed:

- The coefficient of variation (CV) of pulse period >25%.
- The difference between an adjacent pulse period ≥0.14s and the number of these pulse measurements is more than 53 percentage points of the total number of pulse measurements.

Please do not use any other cuff other than that supplied by the manufacturer, otherwise it may cause a biocompatibility hazard and might result in measurement errors.

The monitor might not meet its performance specifications or cause safety hazard if stored or used outside the specified temperature and humidity ranges in specifications.

Please do not share the cuff with other infective person to avoid cross-infection.

The device is not intended for use on babies, children or pregnant women (Clinical testing has not been conducted on babies, children or pregnant women).

Motion, trembling and shivering may affect the measurement reading.

The device should not be used for patients with poor peripheral circulation, noticeably low blood pressure or low body temperature (there will be low blood flow to the measurement position).

The device should not be used for patients who use an artificial heart and lung (there will be no pulse).

Consult your physician before using the device for any of the following conditions: common arrhythmias such as atrial or ventricular premature beats or atrial fibrillation, arterial sclerosis, poor perfusion, diabetes, pre-eclampsia, renal diseases.

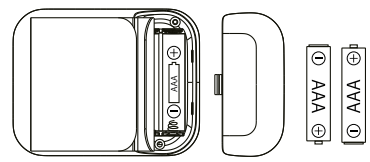
This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Please note that changes or modification not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Swallowing batteries and/or battery fluid can be extremely dangerous. Keep the batteries and the unit out of the reach of children and those with a disability.

8 SETUP AND OPERATING PROCEDURES

1 Loading the battery:

- Open the battery cover at the back of the monitor.
- Load two "AAA" size batteries. Please note polarity.
- Close the battery cover.

When LCD shows battery symbol , replace all batteries with new ones. Rechargeable batteries are not suitable for this monitor.

Remove the batteries if the monitor is not to be used for a month or more to avoid relevant damage caused by battery leakage.

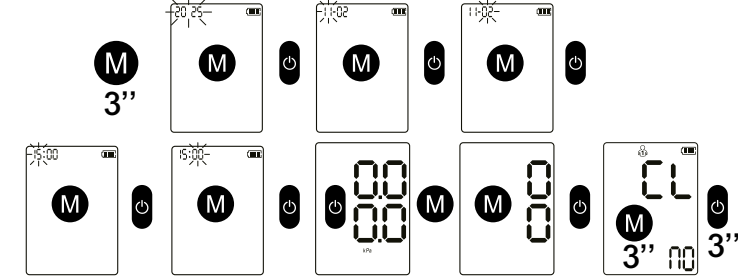
Avoid battery fluid getting into your eyes. If it should get into your eyes, rinse immediately with plenty of clean water and contact a physician.

The monitor, the batteries and the cuff must be disposed of according to local regulations at the end of their usage.

2 Time, date and unit conversion setup :


The WBPM-200 automatically stores up to 199 measurement values for each user. The memory function stores the measurement values with the date and time. Set the monitor to the current date and time before taking any measurement for the first time. The date and time cannot be changed when taking a measurement.

- In the power-off mode, keep pressing the « M » button for 3 seconds, and the year will flash to enter the setting mode.
- Press the « M » button again to adjust the year, then press the « ⌂ » button to save your setting and enter the month setting mode.
- Follow the same steps to adjust the month/hour/minute.
- Finally, press the « ⌂ » button to switch mmHg/kPa. mmHg is set by default. Press the « M » button to confirm the selection. Then press the « ⌂ » button again, press the « M » button briefly to select the user group, the measured values of which need to be deleted. Then keep pressing the « M » button for 3 seconds to delete the memory of the selected user and the «  » icon will appear on the screen. You can stop the settings by pressing the « ⌂ » button for 3 seconds to exit the current settings and turn off the device.

3 User mode :

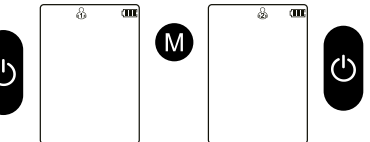In power-off mode, press the « ⌂ » button and the user icon will flash. Then press the « M » button to switch users. Finally press the « ⌂ » button again to confirm the user.

4 Connecting the cuff to the monitor :

The cuff is attached to the monitor when in its packaging. Should the cuff become detached, align the two plugs and four brackets of the cuff with the monitor's plug sockets and bracket sockets and press the cuff on to the monitor until the plugs and brackets are securely attached.

5 Taking your blood pressure reading :

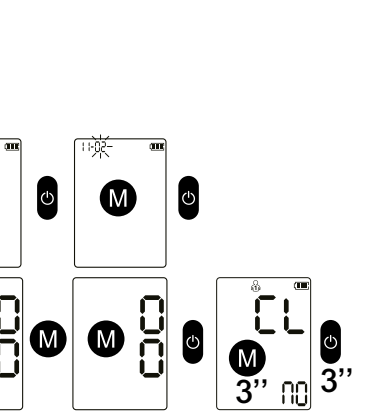
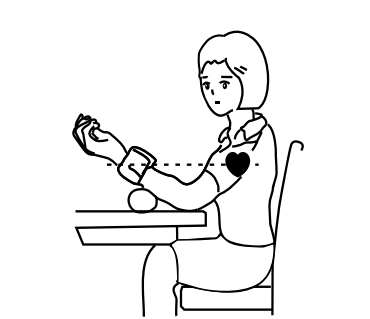
- Place the cuff around the bare left wrist 10-15mm above the wrist joint on the palm side of the wrist. Fasten the cuff using the Velcro strip.
- While seated, place the arm with the cuffed wrist in front of your body on a desk or table with the palm up. If the cuff is correctly placed, you can read the LCD display.
- The cuff must be neither too tight nor too loose.

Note :

- Please refer to the cuff circumference range in "SPECIFICATIONS" to make sure that the appropriate cuff is used.
- Do not move your arm, body or the monitor during measurement.
- Stay quiet and calm for 5 minutes before measuring your blood pressure.
- Make sure you keep the cuff clean. Clean the cuff using a soft wet soft cloth and mild detergent if the cuff becomes dirty. Do not remove the cuff from the monitor. Cleaning the cuff after it has been used 200 times is recommended. Do not place the cuff around your arm if your arm shows any inflammation, has any acute diseases, infections or skin wounds.


6 Body posture during measurement :

- Be seated with your feet flat on the floor, and do not cross your legs.
- Place palm upwards in front of you on a flat surface such as a desk or table, with your elbow resting on this flat surface.
- The middle of the cuff should be at the level of the right atrium of the heart.

7 Taking your blood pressure reading :

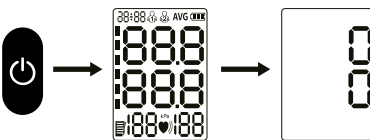- After applying the cuff and your body is in a comfortable position, press the « ⌂ » button. After all icons are turned on, the monitor will start inflating for measurement and display 00.

Check the measured values after the measurement is complete.

8 Memory function :

- Each measurement value will be stored automatically after measurement, and the monitor

can store up to 199 sets of values for each user. When the memory is full, the old measurement value will be covered by the new one.


- Deleting th memory : press the « M » button, press the "M" button briefly to select the user group, the measured values of which need to be deleted. Then keep pressing the « M » button for 3 seconds to delete the memory of the selected user and the «  » icon will appear on the screen.

Note : Please consult a health care professional for interpretation of pressure measurements.

9 Assessing high blood pressure for adults

The following guidelines for assessing high blood pressure (regardless of age or gender) have been established by the World Health Organization (WHO). Please note that other factors (e.g. diabetes, obesity, smoking, etc.) need to be taken into consideration. Consult your physician for an accurate assessment and never change your treatment by yourself.

Range of blood pressure values	Systolic (in mmHg)	Diastolic (in mmHg)	Value displayed
Level 3: acute hypertension	≥ 180	≥ 110	3
Level 2: intermediate hypertension	160 – 179	100 – 109	2
Level 1: mild hypertension	140-159	90 – 99	1
Normal hight	130 - 139	85 - 89	N
Normal	120 - 129	80 - 84	N
Optimum	<120	<80	N

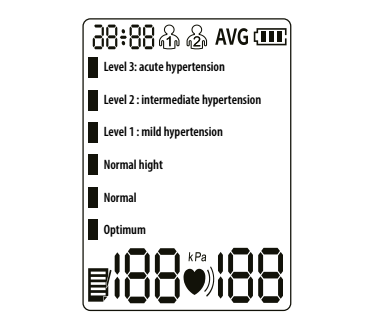10 Error indicators

When the measurement is abnormal, the following symbols will appear

Error indicators	Cause/ Correction
Er U	Inflation cannot reach 30 mmHg in 7 seconds.
Er H	Inflation reaches 295 mmHg.
Er 1	The pulse rate is not detected correctly.
Er 2	Moving, talking, or any magnetic disturbance during a measurement.
Er 3	The test result is abnormal.
Er 23	SYS value is lower than 45 mmHg.
Er 24	SYS value is higher than 230 mmHg.
Er 25	DIA value is lower than 25 mmHg.
Er 26	DIA value is higher than 160 mmHg.

11 Troubleshooting

Anomaly	Inspection items	Countermeasures
No display after the power button has been pressed	Whether the battery power is insufficient	Replace the batteries
Unable to measure due to display error	Whether the arm is moved when pressurized.	Keep your arm and body still.
	Whether you talk during measurement.	Keep quiet while measuring blood pressure.
		Please contact the dealer to replace with a new cuff. Do not change the cuff by yourself.
Air leakage of cuff	The airbag of the cuff is ripped.	

If the blood pressure still cannot be measured after trying the above-stated solutions, please contact the dealer. Do NOT attempt to dismantle the device yourself.

9 MAINTENANCE

- Do not drop this monitor or subject it to strong impact**
- Avoid high temperatures and solarization. Do not immerse the monitor in water as this will result in damage to the monitor.**
- If this monitor is stored under temperatures close to freezing, allow it to acclimatize to room temperature before use.**
- The monitor requires 6 hours to warm up from the minimum storage temperature between uses until the monitor is ready for its INTENDED USE when the ambient temperature is 20 °C.**
- The monitor requires 6 hours to cool down from the maximum storage temperature between uses until the monitor is ready for its INTENDED USE**

- Do not attempt to dismantle this monitor.**
- If you do not use the monitor for a long time, please remove the batteries.**
- It is recommended the performance should be checked every 2 years or after repair. Please contact the service center.**
- Clean the monitor with a dry, soft cloth or a soft cloth squeezed well after being moistened with water, diluted disinfectant alcohol or diluted detergent.**

No component can be maintained by the user in the monitor. The circuit diagrams, lists of component parts, descriptions, calibration instructions, or other information which will assist the user's appropriately qualified technical personnel in repairing those parts of equipment which are designated as repairable can be supplied.

The monitor can maintain the safety and performance characteristics for five years and the cuff integrity is maintained after 1,000 open—close cycles of the closing mechanism.

10 SAFETY INSTRUCTIONS

The following symbols appear in these instructions for use :

Warning instruction indicating a risk of injury or damage to health.

Safety instruction indicating possible damage to the unit/accessory.

Note on important information.

	This icon indicates prohibitions (what you should not do). Matters involving actual prohibitions are indicated by text or pictures in or near this icon. The left icon refers to "general prohibition".		
	This icon that is compulsory (what always needs to be observed). Matters involving actual compulsory actions are indicated by text or pictures in or near this icon. The left icon refers to "general compulsion".		
	This icon indicates something cannot be dismantled.		
	Type-BF applied part		Indicates a medical device that needs to be protected from humidity and moisture. The equipment should be sent to the specialist agencies according to local regulations for separating collection after its useful life.
	Keep out of the sun		Indicates a medical device that needs to be protected from humidity and moisture. The equipment should be sent to the specialist agencies according to local regulations for separating collection after its useful life.
	Lot number		Medical device
	Serial number		CE mark
	Please refer to the instruction manual		RoHS mark
	Date of manufacture		Manufacturer
	Vertical upward		EU authorized representative
	The device is protected from water splashes. Water splashed against the enclosure from any direction shall have no harmful effects.		
	Indicates temperature limits within which the transport package shall be stored and handled.		

The patient must follow the doctor's instructions and should not carry out any self-diagnosis or treatment themselves up to the measured result. Self-diagnosis of measured results and treatment are dangerous. The device should not be used to diagnose illness, carry out first aid or continuously monitor measuring. This device cannot be used for patient and surgical care. It can be used in household or set places only. The patient is an intended operator, and all the functions can be safely used. Please press the ON/OFF button to stop the procedure if you feel uncomfortable with the wrist, or if the air is inflating abnormally without stopping.

Caution

E-IM_WBPM-200_LA090208-001-11/12/2024

Do not let a child under 12 years old or anyone unable to express intention use the device. They should be accompanied by a capable adult to avoid accident or problems.

Do not use the device for purposes other than measuring blood pressure to avoid accident or problems.

Do not use mobile phones around the device. Please do not use the device around a magnetic field.

Do not use the device when you are moving
Do not use the device outdoors or in wet rooms.
Do not use the device near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging where the intensity of EM disturbances is high.

Use of this equipment adjacent to or stacked with other equipment should be avoided as it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify it is operating normally.

Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment.
Portable RF communications equipment (including peripherals like antenna cables and external antennas) should be kept at least 12 inches (30cm) from the device and its parts to avoid impaired performance.

Do not dismantle, repair, or remodel the main unit or the cuff of the blood pressure monitor.

Requests from the manufacturer

For any patient, do not measure more than 3 times continuously as there should be an interval of at least 5 minutes' rest between two measurements, otherwise it could cause extravasated blood.

Do not measure your blood pressure more than 6 times each day.

Do not apply the cuff over a wound as this can cause further injury.

When taking measurements the pressure shall not exceed 295 mmHg. If so, press the ON/OFF button to stop.

Do not use force to bend the wrist cuff or the air tube.

Do not knock or drop the main unit.

Only use the accessories specified in the manual. Third-party accessories not approved by the manufacturer may cause faults or injuries.

Dispose of the monitor at the end of its service life by following local regulations regarding the disposal of this product.

When performance changes, such as inaccurate measurement or abnormal display, stop using the device immediately and contact the after-sales service.

Do not leave small parts where children can reach them.

Children may swallow them. If such an accident happens, contact a doctor immediately.

No maintenance while in use.

The user can maintain the product. The maintenance method is described in this manual.

11 WARRANTY INFORMATION

Limited warranty

LANAFORM guarantees this product from any material or manufacturing defect for a period of two years from the date of purchase, except in the circumstances described below.

The LANAFORM guarantee does not cover damage caused as a result of normal wear to this product. In addition, the guarantee covering this LANAFORM product does not cover damage caused by abusive, inappropriate or incorrect use, accidents, the use of unauthorized accessories, changes made to the product or any other circumstance of whatever kind outside LANAFORM's control.

LANAFORM may not be held liable for any type of circumstantial, indirect or specific damage.

All implicit guarantees relating to the suitability of the product are limited to a period of two years from the initial date of purchase as long as a copy of the proof of purchase can be supplied.

On receipt, LANAFORM will repair or replace your appliance at its discretion and will return it to you. The guarantee is only valid through LANAFORM's Service Center. Any attempt to maintain this product by a person other than LANAFORM's Service Center will render this guarantee void.

12 SERVICE CENTER

	Shenzhen AOJ Medical Technology Co., Ltd Room 301&4F, BlockA, Building A, Jingfa Intelligent Manufacturing Park, Xiaweiyuan, Gushu Community, Xixiang Street, Bao'an District, 518126 Shenzhen, PEOPLE'S REPUBLIC OF CHINA
	Share Info GmbH Heerdter Lohweg 83, 40549 Düsseldorf, GERMANY Tel.: +0049 179 5666 509 Email EU-Rep@share-info.com

	LANAFORM Rue de la Légende 5S, B-4141 Louvain-la-Neuve, Belgium Tel.: 0032-4-360-92-91
---	--

13 ELECTROMAGNETIC COMPATIBILITY INFORMATION

Table 1 – Emissions

Emissions test	Compliance
RF emissions CISPR 11	Group 1
RF emissions CISPR 11	Class B
Harmonic emissions IEC 61000-3-2	Not applicable
Voltage fluctuations/flicker emissions IEC 61000-3-3	Not applicable

Table 2 – Electromagnetic immunity

Immunity test	IEC 60601-1-2 Test level	Compliance level
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air
Electrostatic fast transient/burst IEC 61000-4-4	Not applicable	Not applicable
Surge IEC 61000-4-5	Not applicable	Not applicable
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	Not applicable	Not applicable
Power frequency magnetic field IEC 61000-4-8	30 A/m; 50-60 Hz	30 A/m; 50-60 Hz
Conducted RF IEC 61000-4-6	Not applicable	Not applicable
Radiated RF IEC 61000-4-3	10 V/m 80MHz - 2.7GHz 80% AM at 1 kHz	10 V/m 80MHz - 2.7GHz 80% AM at 1 kHz

NOTE: U_i is the a.c. mains voltage prior to application of the test level.

Table 3 – Fields of proximity from RF wireless communications equipment

	Test frequency (MHz)	Band (MHz)	Service	Modulation
Radiated RF IEC 61000-4-3 (test specifications for port immunity enclosure to RF wireless communications equipment)	385	380-390	TETRA 400	Pulse modulation 18 Hz
	450	430-470	GMRS 460, FRS 460	FM ±5 kHz deviation 2 Hz sine
	710			Pulse modulation 217 Hz
	745	704-787	LTE Band 13.17	
	780			
	810			
	870	800-960	GSM 800/900, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modulation 18 Hz
	930			
	1720		GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse modulation 217 Hz
	1845	1700-1990	Bluetooth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modulation 217 Hz
Radiated RF IEC 61000-4-3 (test specifications for port immunity enclosure to RF wireless communications equipment)	2450	2400-2570		
	5240			
	5500	5100-5800	WLAN 802.11 a/nt	Pulse modulation 217 Hz
	5785			
	Maximum power (W)	Distance (m)	IEC 60601-1-2 Test level (V/m)	Compliance level (V/m)
Radiated RF IEC 61000-4-3 (test specifications for port immunity enclosure to RF wireless communications equipment)	1.8	0.3	27	27
	2	0.3	28	28
	0.2	0.3	9	9
	2	0.3	28	28
	2	0.3	28	28
	2	0.3	28	28
Radiated RF IEC 61000-4-3 (test specifications for port immunity enclosure to RF wireless communications equipment)	0.2	0.3	9	9
	0.2	0.3	9	9

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