	FR	DE	NL	EN
①	Bouton Marche/Arrêt 3 niveaux de chauffe	Ein-/Ausschalter 3 Wärmestufen	Aan/uit button 3 verwarmingsstanden	On/Off button 3 heating levels
②	Zone de chauffe (10 x 15 cm)	Heizzone (10 x 15 cm)	Verwarmingskussen (10 x 15 cm)	Heating zone (10 x 15 cm)
③	Powerbank 5000 mAh incluse	Powerbank 5000 mAh inkl.	Inclusief powerbank 5000 mAh	Powerbank 5000 mAh incluse
④	Elastiques de resserrage	Gummibänder für Weitenregulierung	Elastische aantrekbanden	Elastics for tightening

CEINTURE DE SOUTIEN CHAUFFANTE

FR

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WARMING RECHARGEABLE BELT

EN

Warning: Please read this instruction manual carefully before using the instrument and keep it for future use.

A. EXPLANATION OF SYMBOLS

This instruction manual belongs to this device. It contains important information about starting up and operation. Read the

instruction manual thoroughly. Non-observance of these instructions can result in serious injury or damage to the device.

Do not chlorinate!

Do not dry the heated cushion in a tumble drier! The cushion cover may not be put in a dryer!

Do not iron or press

Do not dry clean

Do not insert any pins, safety pins or needles

Do not use when folded or creased

Not suitable for use with small children (0-3 years)

B. SAFETY INSTRUCTIONS

Special safety instructions

1. This device must not be used by children under 3 years of age as they are unable to react in the event of it overheating.
2. This device must not be used by small children (3-8 years) unless a parent or guardian has pre-set the necessary controls, or the child has been given instructions on how to use the heating pad safely.
3. This device can be used by children aged 8 or over, and by persons with physical, sensory or mental disability or with limited experience or knowledge, provided they are under supervision or have received instructions on how to use it.
4. Children must not play with the it.
5. Children must not be allowed to clean or maintain this device without supervision.
6. Make sure that you do not fall asleep while using this device!
7. If you are very tired while using it, you can select level 1 for your safety.
8. You must examine the device frequently for signs of wear and tear or damage. If you detect signs of wear or tear or damage, or if the device has been used inappropriately stop to use it.
9. The switch mechanism MUST NOT come into contact with water while the product is being washed or dried.
10. Use only with the detachable powerbank provided.
11. Do not use if folded or creased.
12. Do not attach safety pins, needles or pins of any kind to this device.
13. This product is not intended for use in hospitals, clinics etc
14. This is not a medical product.
15. Do not use with a helpless person, child or any person with low sensitivity to heat.

General safety instructions

1. The device is intended exclusively for private and not for commercial use. Only use this device as describe in this instruction manual.
2. Do not handle the unit or plug with wet hands.
3. When disconnecting the cable, always pull at the plug; do not pull on the cable itself.
4. Never try to repair it by yourself. It must be repaired by the manufacturer or its service agent or a similarly qualified person in order to avoid an electric shock.
5. Always place your device on a dry surface.
6. Do not use outdoors.
7. Do not let the power cord on the edge of a table or work surface, and prevent it comes into contact with hot surfaces.
8. Keep it away from heat sources such as radiators, to avoid deformation of plastic parts.
9. Do not touch the hot surface. Keep the appliance out of reach of children.
10. This unit is not designed to be operated by means of an external timer or a separate remote control system.

C. HOW TO USE IT

1) Charging the battery

Before first use, connect one end of the power cable to the powerbank and the other end to a USB port on your computer or to an AC adapter (not included).

The powerbank's lights will flash while charging. When the charge is complete (about 3 hours), disconnect the cable to extend the life of the battery.

You will be able to use your device for about 60 to 90 minutes before the next recharge.

2) Starting up

- Position the belt around the waist, making sure to place the heating zone ② on the painful or tense back area.

C 0105 / xxx

- Then close on the front by adjusting the wide Velcro strap to your waist size and adjust the tightening elastics ④ according to the fit that suits you.
 - Connect the cable from the belt to the powerbank.
 - Slide the powerbank into the pocket provided.
 - To turn on the device, press and hold the Power button ① for at least 3 seconds. The button will flash red.
 - Use the ① button to select the heating level (level 1 = mild; level 2 = medium; level 3 = intense).
 - o Press 1 time: level 1 (40°C) the indicator lights up white
 - o Press twice: level 2 (50°C) the indicator lights up blue
 - o Press 3 times: level 3 (60°C) the indicator lights up red
- When the red light is flashing, if you do not select a mode, the light stops flashing after 5 to 7 minutes and turns white: the belt automatically switches to level 1 (40°C)
- To turn off the device, press the On/Off button for about 3 seconds ①

The belt will quickly give you a feeling of light beneficial heat to relieve pain without blocking your movements.
Its soft and comfortable material is suitable for all body types.

To be worn as soon as the pain appears.

To be used directly on the skin or on a light garment.

AFTER USE

When the device is not in use, unplug the powerbank. If the device is not used for a long period of time, store it in a cool, dry place.

WARNING - RISK OF BURNS

Prolonged use of the device at full power can cause burns.

To avoid skin burns, check the temperature before use with the back of your hand (not the palm of your hand as it is less sensitive than other parts of the body).

D. CARE AND MAINTENANCE

- 1) Unplug the powerbank
- 2) Clean the product only with a dry or slightly damp, lint-free cloth.

Do not use abrasive cleaners to clean the device.


CAUTION

IMPORTANT: Neither the powerbank nor the power cable should come into contact with moisture, water or any other liquid.

If it does, there is a risk of electric shock. Use only when the device is completely dry.

NEVER USE THE DEVICE WHEN IT IS WET OR DAMP!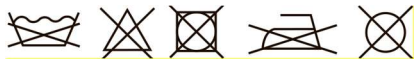

E. FEATURES

- Textile material: 100% polyester
- Power supply : powerbank 5000mAh
- Autonomy : about 1h30
- 8W – 5V

The European directive 2012/19/EU about electronic and electric waste, requires that you can't throw away defective domestic appliance with common waste. Used device has to be collect separately to optimize the recuperation rate and the recycling of the materials to reduce the impact on the health and the environment.

Precision and precautions about the battery use

- o Throwing away a used battery in the environment or with household waste pollutes and avoids the recuperation of reusable materials.
- o It is important to limit the battery consumption and to respect the following rules:
 - Favor alkaline battery (last longer than saline battery) and when it's possible use rechargeable battery.
 - Leave used batteries and accumulators in adapted containers that you can find in some retailers. In that way, metals don't pollute the environment and the health.
- o Battery has to be inserted by respecting the polarity show on the apparel and the battery. An incorrect positioning can damage the apparel, cause leaking and even provoke fire and the explosion of the battery.
- o To assure a good functioning, battery has to be in a good condition. If there is a problem, put new battery.
- o Never try to recharge battery not made for this. It could leak, warm up, cause fire or explode.

C 0105 / **xxx**

- *Replace all the battery at the same time. Never mix up alkaline battery with saline battery.*
- *Used battery has to be removed.*
- *Also, remove the battery if you're not using the apparel for a long time.*
- *Never try to short-circuit the terminal of a battery.*
- *Never throw away the battery in fire because they can explode.*
- *Take out the accumulator from the apparel before reload it.*
- *We recommend that an adult watch if a child change the battery to respect those rules.*
- *If you swallow a battery, go immediately to your doctor or the hospital.*
- *For the best ongoing performance, we recommend using only fresh, high-quality, alkaline batteries.*

Importé par / Importiert von / Geïmporteerd door / Imported by :
CAPTELEC, 59170 Croix - FRANCE