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Warning: Please read this instruction manual carefully before using the instrument and keep it for future use.

A. SAFETY INSTRUCTIONS

Never leave product unattended during heating.

Do not put the maximum power and do not use the grill function of the microwave.

Do not reheat in gas oven or electric oven.

Always check the temperature with your hand to prevent burns.

Allow the product to cool at least 1 hour between each use.

For children, let cool for 5 minutes before applying.

In case of blood circulation or cardiac diseases ask your doctor for advice.

Do not ingest components. Keep away from young children.

Reheat only from room temperature.

DO NOT OVERHEAT - WARNING: THIS PRODUCT MAY CAUSE BURNS: In case of overheating, leave the products.

B. HOW TO USE IT

HOT

Place your heated item in the microwave oven and heat for 60 to 80 seconds at medium temperature (max. 800W), testing in 30 second intervals to reach the desired temperature. Then place the shoulder / neck cover at your convenience for muscle relaxation, joint pain, false movement and to relieve stiffness and cramps.

Never exceed the recommended heating time!

Indeed, seeds heated too heavily, too long or too frequently in a short period of time, could by the level of accumulated heat and drought take the risk of burning you, or even in extreme cases, catch fire by combustion.

Flax seeds are a natural product of this fact, when they are reheated, they can release moisture during the first use. This is a natural phenomenon that fades, with each use. So when you first use it, we recommend drying the moisture with a towel.

Always check for dryer or other seed items in the microwave. Observe strictly the operating instructions.

COLD

Protect your item in a waterproof plastic bag and place it in your freezer for at least 1 to 2 hours or more if necessary.

Remove it from the freezer check the temperature, remove it from the plastic bag. Then place the shoulder / neck cover at your convenience to relieve a hematoma or a torticollis for example.

C. CARE AND MAINTENANCE

Clean with a damp sponge (never immerse or underwater) - Do not iron - If the product is removable, you can clean the cover (without the trim) by machine.

D. FEATURES

100% polyester cover

Filling: rice seeds, lavender, peppermint, cinnamon and eucalyptus

This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capacities, or by persons lacking in experience or knowledge, unless they are supervised or instructed on the use of this apparatus by a person responsible for their safety. Children should be supervised to ensure they do not play with this appliance.

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