	FR	EN
①	Masque facial	Face mask
②	Masque nasal	Nasal mask
③	Gobelet doseur	Measuring cup
④	Support aromathérapie	Aromatherapy support
⑤	Réglages	Settings
⑥	Base	Base

INHALATEUR ELECTRIQUE

FR

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ELECTRIC INHALER

EN

Warning: Please read this instruction manual carefully before using the instrument and retain it for future use.

A. SAFETY INSTRUCTIONS

1. This appliance is intended for household use only. Use the appliance only as described in this manual.
2. Before plugging in the appliance, make sure that the electrical voltage in your home corresponds to that indicated on the appliance's rating plate.
3. To avoid electric shock, never immerse the power cord, plug or any other part of the appliance in water or any other liquid.
4. Do not leave the appliance unattended when plugged in.
5. Do not handle the appliance or the plug with wet hands.
6. Always unplug your appliance after each use and allow it to cool before cleaning, storing or moving it.
7. When unplugging the cable, always pull it by the plug, do not pull on the cable itself.
8. When using the appliance, always unwind the power cord completely from its housing to avoid overheating the appliance.
9. Do not use the appliance if the power cord or plug is damaged, if it malfunctions, if it has been dropped or damaged in any way, or if it has been damaged in any way.

10. Never attempt to repair the appliance yourself. Always have it repaired by a service center or a qualified repair center.
11. The power cord should be checked regularly to avoid any risk of damage. If the power cord is damaged, it must be replaced by a qualified repair center to avoid any danger.
12. Always use the appliance on a dry surface.
13. Do not use the device outdoors.
14. Do not let the power cord hang over the edge of a table or work surface, or allow it to come into contact with hot surfaces.
15. Keep the device away from heat sources such as radiators, to avoid deformation of the plastic parts.
16. Do not use abrasive cleaning products to clean the appliance.
17. This appliance is not intended to be operated by means of an external timer or by a separate remote control system.
18. This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or persons lacking experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety. Children should be supervised to ensure that they do not play with the appliance.

B. **WARNING**

1. Do not use this product if you have heart disease, severe rosacea, eczema or psoriasis.
2. Pregnant women or people with serious health conditions should consult a doctor before using the product, especially when adding herbs or essential oils.
3. Discontinue use immediately if you experience any skin irritation or discomfort while using essential oils or herbs.

C. **USE**

1. Fill the base ⑥ with water: Pour 40 ml of water into the base. Do not fill above the "MAX" line indicated inside the filling area.
2. Attach the face mask ①: Align the bottom of the steam mask with the slot on the base ⑥, then rotate it clockwise to lock it in place.
3. Using the nasal mask ②: If you use the nasal mask, place it directly in the center of the face mask and adjust the steam direction as needed.
4. Plug in the inhaler and select the desired steam setting ⑤:
 "L" for low steam
 "H" for high steam
5. Turn off the appliance after use: once the steam has evaporated or when you have finished using it, turn off the appliance and unplug it from the electrical outlet.
6. Post-steam routine: After enjoying the steam, relax for 8 minutes, then wash your face with cold water to close your pores.
7. Aromatherapy: Open the aromatherapy holder ④, place a cotton pad soaked in essential oil, then close the holder and place it at the base ⑥.

Note: Keep a distance of at least 10cm between the inhaler and your face/nose.

D. **MAINTENANCE ADVICE**

Clean the product only with a dry or slightly damp, lint-free cloth.
 Do not use abrasive cleaning products to clean the appliance.

E. **TECHNICAL CHARACTERISTICS**

- Materials: PC, ABS, PP
- Mains power supply: 220 V 50HZ 100 W (130 cm cable)
- Tank capacity: 40 ML
- Accessories included: facial tip, nasal tip, aromatherapy holder, measuring cup

Importé par / Importiert von / Geïmporteerd door / Imported by / Importato da / Importado por :
CAPTELEC, 59170 Croix – France / www.captelec.fr

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