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@BoardingRing

Welcome on board !

Boarding Glasses prevent and mitigate motion sickness, whatever the vehicle, for everyone aged 10 and over. These glasses use patented **Boarding Ring®** technology, thanks to their **signature blue liquid**, which reacts to accelerations and provides vision with an inertial reference.

How to use Boarding Glasses ?

Put on Boarding Glasses **as soon as the first signs appear.**

Your senses re-synchronize in **a few minutes**

When **all is good**, you can take them off and enjoy the trip !

You wear glasses ?

Fits over most of the prescription glasses. If it doesn't, remove your glasses and wear Boarding Glasses.

Special precautions :

- If you feel no rapid improvement, **move the glasses forward** 1 cm towards the tip of the nose.
- Simultaneous use of motion sickness drugs, or any other psychoactive treatment, **degrades the effectiveness** of Boarding Glasses.
- “A few minutes” is an estimation, **wear Boarding Glasses as long as you need them.** For the most sensitive ones, it's advisable to wear Boarding Glasses as a preventive measure, as soon as you start your journey.
- Make sure the **light level** is sufficient to see the blue liquid.
- An **adaptation** period (2-3 uses) may be necessary before observing a significant effect.
- **If no significant improvement is noticed**, please contact us at glasses@boardingring.com.
- The blue liquid is a neutral solution suitable for food contact. In case of contact with skin, mouth or eyes, rinse with water. Consult a doctor if a reaction occurs.
- This model is not a pair of optical glasses or sunglasses.
- Not suitable for direct observation of the sun.
- Not suitable for protecting against artificial light sources.
- Not suitable for protecting the eye against mechanical hazards.
- Driving with these glasses is not recommended due to field of view reduction.
- When unused, avoid direct sunlight on your glasses.
- After use, store your glasses in their case, and protect them from extreme temperatures.
- Clean your glasses with a microfiber cloth, and soap and water if necessary.

Complies with ISO 12312-1:2013 / 1:2015 and ANSI Z80.3-2018 standards - Made in France

- Category 0 lenses • Limited UV protection
- Very limited reduction of solar glare

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